



Kera•Soft

General Information

Kera•Soft is a softening agent for soaking tissue blocks prior to sectioning. This softening agent facilitates the mechanical process of sectioning, and the re-hydration effect allows tissue to be cut rather than fracture. Kera•Soft is the re-hydration agent you should routinely use for improved staining, particularly with small tissue biopsy specimens.

Packaging

Catalog #	Volume
1037	500mL
1039	1 L
1040	1 gallon
1043	40mL Transpak
1044	60mL Transpak

Procedure

1. Remove tissue from fixative and wash gently for 1 minute in running water.
2. Place in Kera•Soft for 30 minutes.
3. Wash gently in water for 5 minutes.
4. Place on processor for routine processing

*Following processing and paraffin embedding, tissue can be "face" softened by placing cut surface of tissue in Kera•Soft for 5 minutes if necessary.